

# Feline Blood Typing™


## Quick Guide

1 Take blood sample in EDTA test tube and measure the hematocrit.

2 Remove cartridge from pouch and insert into analyzer.  
Follow the on-screen instructions.

3 When asked to select hematocrit level, enter the measured hematocrit on the on-screen keyboard.  
Based on the hematocrit input, the analyzer will calculate the amount of blood needed to be added to the diluent tube.

4 Add the amount of blood displayed on the screen to the diluent tube using a 100  $\mu$ L fixed volume pipette.

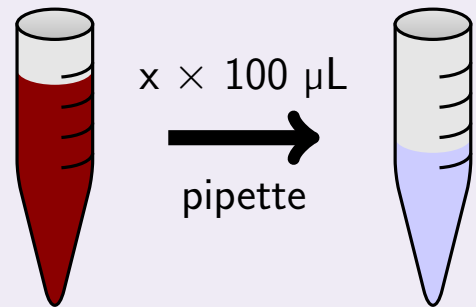

5 Mix diluent and blood using the pipette by aspirating minimum 5 times.


6 When prompted on-screen, extract 100 micro liter sample from the tube using the fixed volume pipette and deposit the sample into the well on the cartridge.

7 Discard pipette tip and wait for result.


## Materials supplied



Cartridge

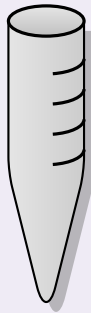

Diluent tube

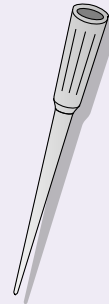

Pipette tip