


# PT/aPTT Coag Combo™

## Quick Guide

1 Take blood sample in citrate test tube.



2 Insert a new QuickVet® cartridge into the cartridge slot.

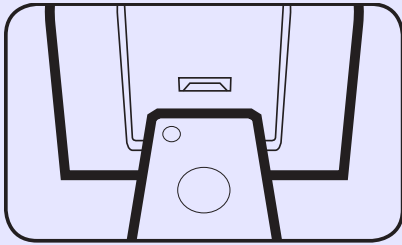


3 Use the on-screen keyboard to enter the 7-digit cartridge code.

4 Confirm species by touching the appropriate button.

5 Use the on-screen keyboard to enter patient and sample ID.

6 Start the test by touching the "Start Test"-button.

7 When the message "Add blood and wait" is displayed on the screen add a droplet of blood to the sample well using the supplied pipette.

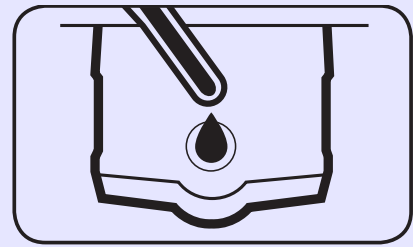

8 Test result is displayed on the screen after a few minutes. Read result.